

SAMARIT – MOBICARE

Therapeutical smooth bed cover

SAMARIT Mobicare is used for Decubitus Prophylax enabling to reposition longtime bedridden patients every two hours from back- to side position without any forces. The patient will be pulled to the edge of the bed on the Mobicare, turned to the center of the bed in a side position. A bolster cushion in the back and if possible one leg in a slight angle will increase the safety of the patient

The innercore of the Mobicare bed cover consists of highly-elastic polyester fibers. The cover has a tubular structure with a easy gliding nylon fabric inside. Half of the outer cover consists of sanforized cotton material which is gentle on the skin, the other half consisting of a water-impermeable but micro fibre fabric.

USE

Mobicare is easy to use. It is timesaving and convenient for both nursing staff and for the patient.

- Mobicare is normally placed on the bed with the white cotton fabric facing upwards.
- The same technic is used to bring the product under the patient as you apply for a bedsheet
- Mobicare is placed on the bed with the water-proof special fabric (yellow) facing upwards for incontinent patients.
- Before the patient assumes the sitting position in bed, a fixing strap can be pulled through the upper loop and secured to the bed frame.

ADVANTAGES

- The patient can be easily moved in the bed without the need for force.
- Daily nursing work is simplified.
- Patients confined to bed for a long period can be comfortably positioned.
- Self-mobilization is possible for paralyzed patients.
- Decubitus is prevented.
- Washable in a washing machine (5 kg).

CLEANING

The smooth bed cover can be disinfected and washed at temperatures of up to 80° C.