

SAMARIT GLIDEBELT (S/M/L)

Art No. 450.2100 small 72-90 cm / 28-35 inch UDI/GTIN 7640144461041	Art No. 450.2200 medium 88-106 cm / 35-42 inch UDI/GTIN 7640144461058	Art No. 450.2300 large 104-122 cm / 41-48 inch UDI/GTIN 7640144461065
---	---	---



SAMARIT Glidebelts are produced in Europe (EU) and made from solid textiles.

FEATURES AND BENEFITS

- Patient handling belt suitable for patients measuring 72–122 cm around the waist (S / M / L)
- Breathable inner fabric and padding to maximize patient comfort
- Superb inner-skin anti-slip fabric grips the user's clothing, providing additional support to ensure confidence
- Comfortably shaped webbing handles designed for hands of any size
- Adjustable handle position
- Choice of vertical and lateral webbing handholds for easy client support where needed
- Great support to the user's lumbar region

USED BY

- Those who need minor assistance from a single carer, when transferring patient in sitting/standing position (increases security)
-

SECTORS

*Home care

*Occupational therapy

*General wards

*Social services

*Community nursing

*Renal units

INSTRUCTIONS FOR USE

The product is used for the security of the patients during a **seated or standing** transfer, sitting or standing on a Glideboard or Turnplate. The assisted transfer can be made by an educated or instructed carer.

1. Put the 'Glidebelt' around the hip of the patient and fix with the velcro.
2. Secure the 'Glidebelt' with the buckle of the belt and tighten.
3. To hold the patient while transferring (or turning) enter with your hands the appropriate handles. (3-5 on the backside of the Glidebelt)
This permits secure guiding and prevention of damage to the skin of the patient.
4. The carer must follow correct, current manual handling techniques.



Attention: In case of an emergency let the patient slowly slide on a bed, chair, floor.

MATERIAL

Special double coated polyethylene tissue: t e a r – r e s i s t a n t

DIMENSIONS

SIZE: **Small:** 72 - 90 cm

Medium: 89 - 106 cm

Large: 104 - 122 cm

CLEANING

Can be washed at 75°C and must be placed in a **textile bag** when a machine is used (avoid damage).

or

can be cleaned best with "Incidin® OxyWipe" by ECOLAB or any comparable hospital disinfectant (**no chlorine, no concentrate, no bleach**)



safe patient moving & handling

SAMARIT MEDICAL AG

GEWERBESTRASSE 12 • CH-8132 EGG

TEL: +41 (0)44 918 10 11 • FAX: +41 (0)44 918 24 39

e-mail: info@samarit.com • www.samarit.com

GUARANTEE

2 years

MANUFACTURER

SAMARIT MEDICAL AG

Gewerbestrasse 12, 8132 Egg b. Zürich, Switzerland

Website: www.samarit.com

Contact: info@samarit.com