

## SAMARIT MEDICAL AG

GEWERBESTRASSE 12 • CH-8132 EGG

TEL.: +41 (0)44 918 10 11 • FAX: +41 (0)44 918 24 39

e-mail: info@samarit.com • www.samarit.com

## **Disposable Covers for Samarit Rollbord**

## INSTRUCTION FOR USE

- 1. Dispo Covers are sold in a solid box which holds four rolls. Each roll has a length of 100 meters (3940 inch) and is perforated all 2 meters (79 inch)
- 2. Place the roll in the dispenser or place it on a appropriate shelve.
- 3. Unroll the material in a length of 2 m and take the first length at the perforation.
- 4. Enter the 'Rollboard' into the tubular cover (60 cm/24 inches wide, 200 cm/79 inches long)
- 5. Pull the Dispo Cover over the 'Rollboard' tight on one side and fold the overlapping material on your side over the top of the Dispo cover.
- Press the air out from the overlap, middle to the left and right. This way the cover is tight on the Rollboard.
- 7. Press the air out from the overlap, middle to left and right. This way the cover is tight on the Rollboard.
- 8. <u>Transfer the patient</u> according to the instructions.



- 9. To take of the PE-Cover, you can pull the Rollboard out (2 person) or you pull one side over the other this way you keep the dirt inside and dispose the used cover.
- 10. Clean the used 'Rollboard' with the Ecolab Incidine Oxywipes S or similar disinfection wipes.
- 11. Control that the 'Rollboard' has no damages and keep if ready for next use.
- 12. In case you find a problem take two pictures (one from near, one from far to see the whole product) and ask for assistance.

www.samarit.com, info@samarit.com

Samarit Medical AG, Egg, September 2019