WARRANTY INFORMATION see page 24

Rollbord™

- Instructions for Use** (EN)
- Gebrauchsanweisung** (DE)
- Mode d'emploi** (FR)
- Manuale d'uso** (IT)
- Instrucciones de uso** (ES)

For other languages please look for more translations on our website.

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MD

product name & ID:

see Table 1: Rollbord™ models and UDI-DI page 3

class I MDR 2017/745

GMDN code 15-693

EMDN code V080504

Contact information:

Manufacturer, Support Center, Headquarter

Samarit Medical AG, Switzerland

Gewerbestrasse 12, CH-8132 Egg

Tel.: +41 44 918 10 11

info@samarit.com - www.samarit.com

SRN **CH-MF-000016280**

EC REP

Authorized Representative

Samarit Medical Produkt GmbH

Im See 19, D- 79790 Küssaberg, Germany

Tel.: +41 44 918 10 11

info@samarit.com - www.samarit.com

SRN **DE-AR-000014694**

Incident and Adverse Event:

For any incident or event, please contact immediately:

An Adverse Event Report form is systematically provided to the customer through the company website or per email.

incident@samarit.com

+41 44 918 10 11

Symbols for Cautions and Warnings

Symbol	Symbol explanation
	Caution related to important information potentially leading to misuse, use error, malfunction or damage of the product
	User or patient information: notice related to useful information and recommendation for an efficient use of the system.

Copyright statement

© 2017-2022 All rights reserved by Samarit Medical AG, Switzerland. Products are subject to change without prior notice. Samarit Medical AG, Switzerland provides the information in this booklet to be read by anyone, but retains the copyright on all text and graphics.

Rollbord™ is a registered trademark of Samarit Medical AG, Switzerland.

Table of Content

Table of Rollbord™ models	3
(EN) Quick Instructions	4
(EN) Users Guide	5
(DE) Kurzanleitung	8
(DE) Gebrauchsanweisung	9
(FR) Instructions rapides	12
(FR) Guide de l'utilisateur	13
(IT) Istruzioni rapide	16
(IT) Manuale d'uso	17
(ES) Breves instrucciones	20
(ES) Manual de instrucciones	21
Limited Warranty	24

Table of Rollbord™ models

Modell	REF	UDI UDI-DI
Rollbord™ Hightec		
Standard 178 x 50 cm foldable	440.0100	0764 014 4460013
ICU-Bariatric 178 x 50 cm not foldable	440.0200	0764 014 4460020
Mini-Gyn 91 x 50 cm not foldable	440.0300	0764 014 4460037
Surgiboard 120 x 40 cm not foldable	440.0400	0764 014 4460044
MRI-CT 178 x 40 cm foldable	440.0500	0764 014 4460051
Rollbord™ Professional		
Standard 180 x 50 cm foldable	440.2100	0764 014 4460143
Mini 80 x 50 cm not foldable	440.2200	0764 014 4461201
MRI 180 x 42 cm foldable	440.2300	0764 014 4460150
Ambulance 165 x 42 cm foldable	440.2400	0764 014 4460167
Blue Transfer Board		
Blue Long Wide 180 x 50 cm foldable	440.8300	0764 014 4460228
Blue Short Narrow 100 x 42 cm not foldable	440.8301	0764 014 4460235
Blue Short Wide 80 x 50 cm not foldable	440.8304	0764 014 4460242
Blue Long Narrow 180 x 42 cm foldable	440.8305	0764 014 4460259
Rollbord™ Eco Lite		
EcoLite 180 180 x 50 cm foldable	440.1180	0764 014 4460327
EcoLite 120 130 x 40 cm foldable	440.1120	0764 014 4460334
EcoLite 90 93 x 50 cm not foldable	440.1090	0764 014 4460341
Transglide		
Transglide large 170 x 65 cm	471.1170	0764 014 4460501
Transglide Patglide 152 x 63 cm	471.5263	0764 014 4460518
Transglide small 140 x 48 cm	471.1140	0764 014 4460525
Transglide A & E, 140 x 48 cm	471.1180	0764 014 4461232
Pull Sheet 150 x 150 cm	471.1200	0764 014 4460532

Table 1: Rollbord™ models and UDI-DI

(EN) Quick Instructions

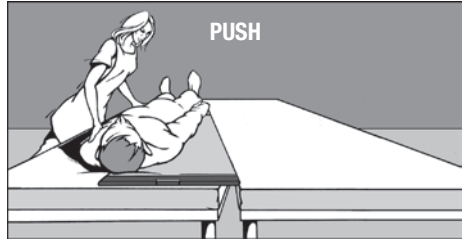
1. Light and convenient


5. Move Rollbord™ and ease patient back


2. Position Rollbord™

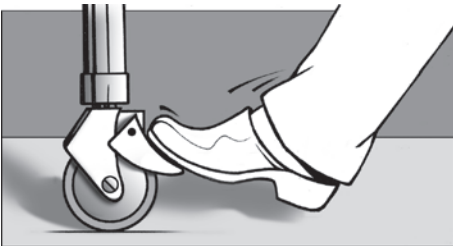6.1 Push patient at hip and shoulder
"One Person Push Method"

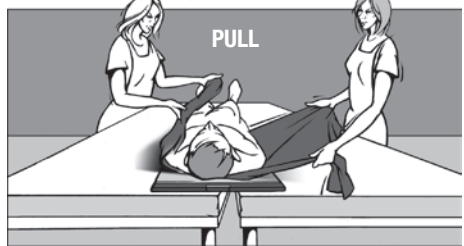3. Lock brakes


6.2 Pull patient over Rollbord™
"Two Person Pull Method"

USA/CA

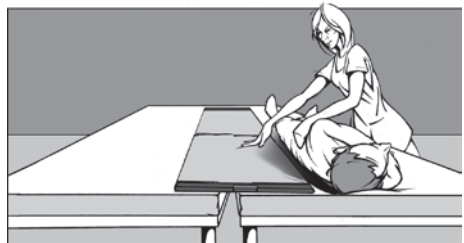4. Turn patient to side

7. Turn patient and remove Rollbord™

(EN) Users Guide

The Samarit Rollbord™ is highly efficient, requiring only 15% of the patient's body weight to either push or pull the patient from one lateral surface to the other. One staff member can easily move an average sized patient with minimal effort.

CAUTION: Do not become over-confident and attempt to move patients above your capacity. With large or touch-sensitive patients, request assistance and use the Two Person Pull method. Always use your proper body position for any patient transfer.

CAUTION: Do not pull Rollbord™ by the handles!!

Picture 1: The Rollbord™ weighs only 6 lbs. (3 kg) and can be carried on the shoulder with the convenient adjustable strap leaving both hands free for stretcher operation.

Picture 2: For easier transfer, ensure that surfaces are leveled or that the receiving surface is slightly lower. Place the Rollbord™ and position next to patient.

Picture 3: Always ensure that brakes are operational and are locked on both vehicles.

Picture 4: Roll the patient towards you and move the Rollbord™ to his back. For tall patients, either place a pillow under shoulder/head and under patient's lower legs to elevate heels.

Picture 5: Ease patient back onto Rollbord™ so that about 1/3 of patient is on the Rollbord™. For wider surfaces, place less of the patient onto the Rollbord™; narrower surfaces more.

„One Person Push Method“

Picture 6.1: Place hands under patient's hip and shoulder. Pushing there positions the centre of patient's weight onto the Rollbord™.

„Two Person Pull Method“

USA/CA

Picture 6.2: Place pull sheet using well-known method under the patient and raise both ends. The patient is then carefully pulled by two persons over the Rollbord™ (preferable method for heavy patients).

Picture 7: Once centered, roll patient towards you and slide Rollbord™ from under the patient with one hand. Pull away side under patient's leg first. Remove Rollbord™ and disinfect, if necessary.

CLEANING

The Rollbord™ can be cleaned best with “Incidin® OxyWipe” by ECOLAB or any comparable hospital disinfectant (**no chlorine, no concentrate, no bleach**). For anticipated soiled conditions use the Rollbord™ disposable cover. It covers ends of Rollbord™ and allows any residue to be folded untouched into the cover, leaving a clean surface.

For replacement of the silver cover, simply unscrew one cap-end and exchange cover. Repairs should only be carried out by authorised personnel.

Do not dispose!

Safety instructions

The user is responsible for reading the User's Guide accompanying every product delivered (in original box). The manufacturer assumes no liability for improper use.

For more instruction consult the training video clip at www.samarit.com

The user must ensure that the product is in perfect working condition to comply with hospital safety regulations.

A Rollbord™ transfer aid that does not meet the required requirements must be removed from service immediately.

For repairs, call our service number or contact your nearest distributor.

Intended use

Lateral transfer of patients without lifting (no lift), gentle repositioning, especially of injured or pressure-sensitive patients, minimized risk of falling.

Spine-friendly repositioning especially for heavy patients for the nursing staff.

Intended medical indication and contraindication

Repositioning from bed or couch to bed or operating table etc.

No contraindication or known non-applicability.

Intended patient population and user profile

For all patients from the age of 10 years, with a patient weight up to max. 160 kg / 180 kg / 240 kg / 310 kg¹, depending on the model.

Applicable in radiology, operating room, ICU, rescue/emergency and resuscitation, etc., depending on the model, size and design.

Use by trained health care professionals.

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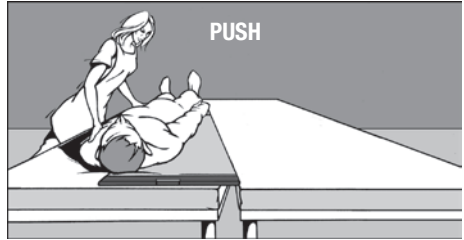
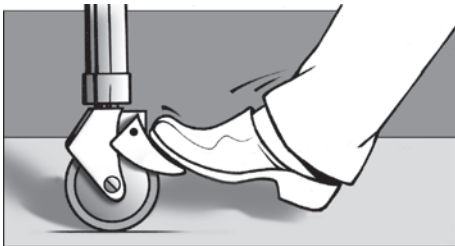
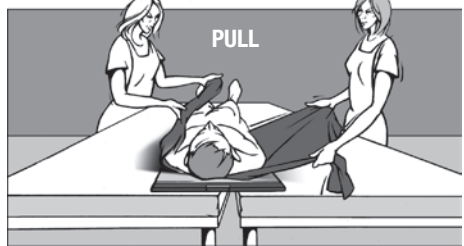
LIMITED WARRANTY

by SAMARIT MEDICAL AG SWITZERLAND

SAMARIT MEDICAL AG Switzerland warrants your product to be free from defects in material and workmanship for a period of 24 / 36 months from the original date of purchase. If you discover a defect in a product covered by this warranty, we will repair at our option using new or refurbished components, or if repair is not possible, replace the item.

Rollbord™ Hightec	36 months warranty
Rollbord™ Professional, Blue Transfer Board, Rollbord™ Ecolite, Transglide	24 months warranty

EXCLUSIONS

This warranty covers defects in manufacturing discovered while using the product as recommended by the manufacturer. The warranty does not cover loss or theft, nor does coverage extend to damage caused by misuse, abuse, unauthorized modification, improper storage conditions, lightening, or natural disasters. The warranty does not cover parts that are subject to normal wear and tear replacement requirements, such as.

LIMITS OF LIABILITY

Should the product(s) fail, your sole recourse shall be repair or replacement, as described in the preceding paragraphs. We will not be held liable to you or any other party for any damages that result from the failure of this product. Damages excluded include, but are not limited to the following: lost profits, lost savings, lost data, damage to other equipment, and incidental or consequential damages arising from the use, or inability to use this product. In no case will SAMARIT MEDICAL AG be liable for more than the amount of your purchase price, not to exceed the current list price of the product, and excluding tax, shipping and handling charges.

SAMARIT MEDICAL AG disclaims any other warranties, express or implied.

By installing or using the product, the user accepts all terms described herein.

HOW TO OBTAIN SERVICE UNDER THIS WARRANTY

Copy of invoice with serial no., pictures of product damage send to info@samarit.com.

The cost of shipping to the manufacturer, or authorized repair center or payment of any customs clearance fees or duties are the responsibility of the user.

Return shipping costs to be paid for by SAMARIT MEDICAL AG, only with written acceptance from Samarit.

Samarit Medical AG
Gewerbstrasse 12
CH-8132 Egg, Switzerland

Tel.: +41 44 918 10 11 - Fax: +41 44 918 24 39 - info@samarit.com - www.samarit.com