

# SAMARIT

safe patient moving & handling



For  
**HEAVY**  
use!

## The inexpensive, convenient Patient Transfer Device

Samarit Transglide is the perfect addition to our extensive product range "Rollbord" and "Rollbody". The Transglide has appreciation in **Hospitals, Nursing & Care Homes** and **anywhere patient lateral transfer** is required.

### **SAMARIT** **TRANSGLIDE** LATERAL TRANSFER

**LARGE**  
**PATGLIDE**  
**SMALL**  
**ACCIDENT & EMERGENCY\***

#### Highlights:

- Easy to use by one or two care workers
- Light, weighs only 3,5 kg
- Made of solid, shatterproof material (HDPE), durable antistatic
- Gliding surface
- Easy to clean and acid resistant
- X-ray translucent
- Allows a difference in transfer height of up to 15 cm
- Safety strips (3M) attached to rear
- Extremely good value

May also be used during resuscitation on bed

\*on special order only

**SAMARIT**

Samarit transfer systems

# The quality transfer board with a "high glide" surface.




SWL\*:  
220 kg / 485 lbs

## LARGE

Size: 170 x 65 cm / 67 x 26 inch

Order No. 471.1170

UDI/GTIN: 7640144460501

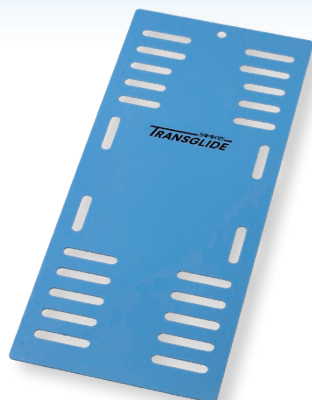

SWL\*:  
220 kg / 485 lbs

## PATGLIDE

Size: 152 x 63 cm / 60 x 25 inch

Order No. 471.5263

UDI/GTIN: 640144460518



SWL\*:  
220 kg / 485 lbs

## SMALL

Size: 140 x 48 cm / 55 x 19 inch

Order No. 471.1140

UDI/GTIN: 7640144460525



SWL\*:  
220 kg / 485 lbs

## ACCIDENT & EMERGENCY

Size: 140 x 48 cm / 55 x 19 inch

Order No. 471.1180

on special order only

## INSTRUCTION FOR USE



Turn the patient to the side using the «pull sheet», push the Transglide under the sheet and return the patient back on to the Transglide. Hold the «pulling sheet» with both hands and keep the back straight and upright.

SAMARIT Products: make your job easier. Additionally, SAMARIT provides the patient with greater comfort and safety.

Transglide is characterized by its simplicity, its durability and its reasonable price.



Using the sheet, slide the patient over the Transglide, maintaining an even motion and keeping the back straight. A second person will assist by pushing gently.

**Cleaning**  
Transglide can be cleaned with the special (Ecolab: Sani-Cloth Active) wipes.

**Environmentally friendly**  
HDPE is easily disposed of and can be recycled. Should a fire occur, no toxic fumes will be emitted.



Once the patient has been placed in the correct position on the bed / couch / X ray table etc, turn him or her to the side slightly. The second assistant pulls the Transglide from the bottom end, easing it slightly, thus removing it from under the patient.

**Convenient storage**  
A notch or indent on the Transglide will facilitate storage on a hook on the wall.



## PULL SHEET

made of durable nylon

Size: 150 x 150 cm / 59 x 59 inch

Order No. 471.1200

UDI/GTIN: 7640144460532

PS: By removing the anti-slide strips the patient can be transferred in a supine or seated position and can therefore remain on the Transglide during treatment (e.g when X-raying).

## High quality standards

SAMARIT transfer aids have:

- CE mark
- Design patents
- 2 years warranty



\*SWL = Safe Weight Limit

[www.samarit.com](http://www.samarit.com)

Download your language from our website or ask your local distributor.

**SAMARIT**  
safe patient moving & handling

SAMARIT Medical AG

Gewerbestr. 12

CH-8132 Egg, Schweiz

Tel.: +41 44 918 10 11 Fax: +41 44 918 24 39

e-mail: [info@samarit.com](mailto:info@samarit.com) - [www.samarit.com](http://www.samarit.com)

**SAMARIT**  
**TRANSGLIDE**  
LATERAL TRANSFER